the desired technique.

ATTENTION:

Before the accomplishment of the molding, wash the area to guarantee the complete removal of the product preventing any interference in the adhesion of the molding materials.

II - In the control of the sulcular fluid:

- Apply a small amount of **RETRACTIVE** in the retractor cord (For ex. Retraflex).
- Insert the retractor cord with **RETRACTIVE**.
- Wash to remove the excess of the hemostatic.
- Finish the restorative procedure.

III - As an assistant in the capillary bleeding interruption:

- 1) - Apply a small amount of **RETRACTIVE** on the area where the hemostasis is desired.
- 2) - After obtaining the hemostasis, wash the area abundantly and check if the bleeding stopped. In case that bleeding did not stop, repeat the procedure.

SPECIAL CARES:

Test the product fluidity previously its use.

Use only the necessary amount and after getting the impression or the stopping bleeding, rinse abundantly to remove any possible residue.

Changes may occur in the gingival tissue coloration due the presence of the ferric in the product composition, which must disappear from 24 to 48 hours.

The product is irritating for the eyes and mucous.

Prevent the contact with the skin and clothes.

The application is topical and must be restrict to the treatment area.

The inadequate use of the product may cause irritation or even some injury of the tissue. Retractor cords impregnated with **RETRACTIVE** should not be forced beyond the apical extension of the gingival crack.

Do not use retractor cords already impregnated.

Do not ingest.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

RETRACTIVE should be not used in patients is known to have sensibility to any of the ingredients listed.

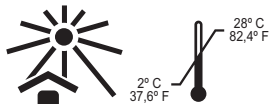
WARNINGS:

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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CE₁₄₃₄

RETRACTIVE Hemostatic for topical use

biodinâmica®

CONTENTS:

1x2,5g syringe **RETRACTIVE**.

3 application tips.

Reg. ANVISA: 10298550103

COMPOSITION:

Ferric sulfate 20%; trickening and deionized water.

INDICATION:

RETRACTIVE is indicated as assistant in cases where the precision molding of gingival border is desirable, in joint action with the retractor cord; helps in the sulcular fluid control, eliminating the contamination, providing an excellent adhesion and reducing the micro-infiltration. The product acts in the bleeding control of capillary information in dental surgical procedures.

TECHNICAL INFORMATION:

RETRACTIVE is a hemostatic for topical use, which contains ferric sulfate in its composition that acts as an astringent and contributes for the temporary gingival removal before the accomplishment of moldings. The ferric sulfate also possess a hemostatic action, assisting in the bleeding interruption of the capillaries and arterioles, producing a fast blood coagulation in the applied region. Considered safe and efficient. The tissues after moved away come back to its normal condition. May be used by cardiac patients due to epinephrine absence.

USE INSTRUCTIONS:

I - As an assistant in cavity preparation moldings:

- Apply a small amount of **RETRACTIVE** in the retractor cord (for ex. Retraflex) and use in accordance to