the correct position.

9. Press slightly (to remain a thin layer of approximately 0,5mm) and remove excess;

10. Light curing:

10.1. Using ceramic brackets light cure during 10 seconds;

10.2. Using metal brackets light cure during 20 seconds (10 seconds each side);

11. Remove the excessive adhesive with bur or appropriate instrumental;

12. The placement of the orthodontic wire can be done immediately after the bracket fixing.

REMOVAL:

- Brackets must be removed with appropriate instrumental;

- Remove the remaining adhesive from the teeth using low-speed handpiece with tungsten carbide bur;

- Do the teeth polishing with a thin pumice powder.

SPECIAL CARES:

To have a perfect adhesion, place a thin layer of **BIOFIX** Light Curing (0,5 mm) between the tooth and the bracket, allowing a total polymerization. A thicker layer can interfere in the result of the adhesion.

After use keep syringes tightly close to avoid exposure to light.

Do not store **BIOFIX** Light Curing near products containing Eugenol. Eugenol interferes in the polymerization.

Avoid contact with eyes. In case of contact, rinse thoroughly with water and seek medical advice if necessary.

Avoid contact with skin. In case of contact, rinse thoroughly with water and soap and consult a doctor if necessary.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

BIOFIX Light Curing should not be used in patients is known to have sensibility to any of the ingredients listed.

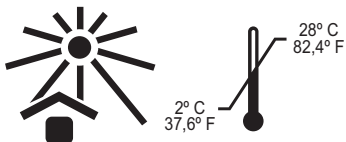
WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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BIOFIX Light Cuting Brackets Fixing Adhesive

CE₁₄₃₄

biodinâmica[®]

CONTENTS:

2x2,5g or 1x4g syringes **BIOFIX** Light Curing and

1x3g syringe Ataque Gel (37% ortho-phosphoric acid);

Reg. ANVISA: 10298550055.

COMPOSITION:

Matrix of methacrylic monomers; inorganic charge; sodium fluoride; initiators; stabilizers and pigments.

INDICATION:

BIOFIX Light Curing is a single component bonding system to fix plastic, metal and ceramic orthodontic brackets to the dental surface.

TECHNICAL INFORMATION:

BIOFIX Light Curing is a special adhesive to fix plastic, metal and ceramic brackets on enamel, there is no need of using the primer.

USE INSTRUCTIONS:

1. Clean the teeth. Do not use any oily product;
2. After prophylaxis, proceed with a relative isolation;
3. Dry the area carefully with air jet, without humidity or oil;
4. Etch enamel surface with 37% orthophosphoric acid (a.e. Ataque Gel) during 30 seconds;
5. Rinse thoroughly the conditioned area with water during 30 seconds;
6. Dry carefully. The presence of humidity in the cleaned surface will not allow the penetration of the Adhesive in the conditioned area
7. Apply a thin layer of **BIOFIX** Light Curing on the bracket base;
8. Immediately after applying **BIOFIX** Light Curing, place carefully the bracket on tooth surface and adjust to