PISTOLA DISPENSADORA

Aplicador

biodinâmica®

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DISPENSER SYSTEM

Application Gun for Dental Materials

biodinámica

CONTENTS:

1x1un. Universal **DISPENSER SYSTEM** for cartridge. Proportion 1:1/ 2:1 and 4:1/10:1.

or

1x1un. **DISPENSER SYSTEM** 1:1/2:1;

or

1x1un. **DISPENSER SYSTEM** 4:1/10:1.

Reg. ANVISA: 10298559008.

INDICATION:

The **DISPENSER SYSTEM** is used for application of dental materials to the base of 2 components presented in cartridge, providing a homogeneous mixer without bubble.

COMPOSITION:

DISPENSER SYSTEM composition: Polypropylene (PP);

EMBOLUS: Polyethylene (PE).

TECHNICAL INFORMATION:

The **DISPENSER SYSTEM** is a manual gun applier, projected to provide an auto mixer of dental materials with base of 2 components, as for example impression material, provisional material, crown and bridges, cements, bleaching materials and others, presented in the cartridge form providing the dispensation in the right proportion and a homogenous mixture.

USE INSTRUCTIONS:

1. Press the embolus lock and pull the **DISPENSER SYSTEM** embolus backward;
2. Up the dispenser lock;
3. Fit in the cartridge in the **DISPENSER SYSTEM** specific proportion;
4. Close the lock until the lock is complete so that the cartridge is properly seated;
5. Insert a compatible mixing tip with the cartridge;
6. Press the dispenser trigger till take way the desired product quantity;
7. For reuse the product, take way the mixer tip used before, fit in a new one and proceed according to the indicated in the item 6.

IMPORTANT: The material polymerized inside the mixer tip shouldn't be removed strength to don't damage the cartridge and the mixer tip and cause leaking.

CLEANING:

- Wash the **DISPENSER SYSTEM** with commercial neutral detergent;
- Do not use organics solvents;
- Remove the past that wasn't polymerized with ethanol (Ethyl Alcohol)
- Remove the product from metallic trays with ethanol (Ethyl Alcohol) or acetone.

SPECIAL CARES:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

WARNINGS:

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

FOR USE IN DENTISTRY ONLY