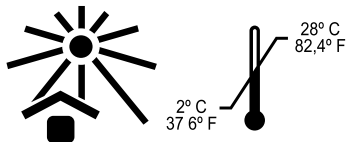
ALVEOLEX

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biodinâmica

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USO EXCLUSIVO DEL DENTISTA

ALVEOLEX

Alveolar Dressing based on Propolis

biodinámica

CONTENTS:

1x10g flask ALVEOLEX

or

2x3g syringes ALVEOLEX

3 application tips.

Reg. ANVISA: 10298550052.

COMPOSITION:

Propolis, Iodoform, Beeswax, Thickener and Vehicle.

INDICATION:

1. Effective for alveolitis prevention;
2. To stimulate the healing after dental extractions;
3. To reduce postoperative pain;
4. In alveolitis treatment;
5. To assist in the healing after teeth extractions on diabetic patients.

TECHNICAL INFORMATION:

ALVEOLEX is specially indicated as a physical barrier after teeth extraction offering protection, once it acts as a moldable alveolar drain plug, forming a solid framework for the tissue regeneration, avoiding contamination of the bone walls. The application is very simple and its paste consistency allows ALVEOLEX to remain in contact with alveolar walls for more time. After placement it requires no special attention from the dentist other than observation of the healing process.

In a period of 30 days, ALVEOLEX will be gradually eliminated from alveolar cavity, as the normal healing of the wound takes place. The presence of Propolis in its formulation helps in the regeneration and healing stimulation as well as act in the analgesia that guarantees a calmer postoperative period to the patient. Iodoform was incorporated to the product to assist and complement the action of the filling by the enhancing of the growth of cells, besides having a light antiseptic power. The Eugenol free formulation causes non-irritation in contact with alveolar tissue.

USE INSTRUCTIONS:

I - In the prevention of alveolitis and to stimulate the healing:

1. After tooth extraction, fulfill the socket with ALVEOLEX and suture.

II - In the already installed alveolitis:

1. Anesthetize by regional blockade;
2. Do the alveolar curettage removing all necrosed tissues;
3. Irrigate with physiologic solution;
4. Provoke bleeding and wait until the socket is full of blood;
5. Fill the alveolus with ALVEOLEX, and proceed the suture.

SPECIAL CARES:

- Avoid the presence of foreign bodies and necrosed tissues in socket before applying the product.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

ALVEOLEX should not be used in patients is known to have sensibility to any of the ingredients listed.

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