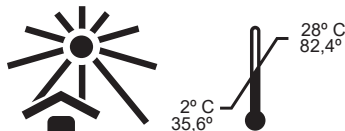
WARNINGS:

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ANVISA nº 10298550087.

COMPOSITION:

37% Orthophosphoric Acid; Methylparaben; Blue Dyeing (C152015); Thickening and Deionized Water.

INDICATION:

For etching enamel and dentin, allowing better retention of composite material used in restoration, attachment, cementing and sealing.

TECHNICAL INFORMATION:

ATTAQUE GEL is an enamel and dentin etching agent used during the bonding procedure of resin materials. When applied to enamel, it causes surface demineralization, which results in micro porosity. In dentin, the acid removes the smear layer, widens the dentinal tubules and exposes collagen fibers. These alterations in the dental tissues allow the penetration of the adhesive and the formation of the hybrid layer, responsible for the micromechanical retention of the material to the dental structure. The blue color and viscosity of **ATTAQUE GEL** facilitate its selective application, only in the desired areas.

USE INSTRUCTIONS:

1. After prophylaxis with pumice stone, wash and dry the area. The use of absolute isolation is indicated;

2. Apply **ATTAQUE GEL** on the area to be etched. Wait 30 seconds in enamel and 15 seconds in dentin;

Caution: Prior pulp protection may be required in deep cavity preparations before the use of **ATTAQUE GEL**.

Fluoride teeth may need a second application;

3. Wash the conditioned area with plenty of water for 30 seconds;

4. Dry the cavity without dehydrating the dentin.

SPECIAL CARES:

- Avoid contact with the skin or mucosa. In case of contact, rinse thoroughly with water.

- After the use, withdraw the syringe pressure by pulling back the plunger.

- During product manipulation, the professional should use gloves and safety glasses;

- Protect the patient with apron and safety glasses.

- In case of contact with the eyes, wash with water, and if necessary, search for medical attention

- Protect the neighbor teeth, with polyester strip, in places where you do not want etch.

- Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

DO NOT INGEST.

CONTRAINDICATION:

ATTAQUE GEL should not be used in patients is known to have sensibility to any of the ingredients listed.