
E.D.T.A. TRISSÓDICO Líquido

Agente Quelante

biodinâmica

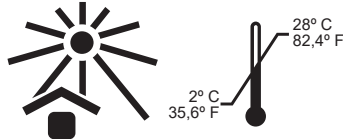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ANVISA nº 10298550072

COMPOSITION:

Ethylene Diamine Tetracetic Disodium Acid, Sodium Hydroxide and Deionized Water.

INDICATIONS:

To facilitate endodontics instrumentation and to improve the quality of posts cementation, removing the smear layer of the interior of prepared conduct.

TECHNICAL INFORMATION:

The **TRISSODIUM E.D.T.A. Liquid** is a widely used product in the preparation of root canal walls, previously to the filling. It acts demineralizing dentinal particles by chelating the ions of calcium and magnesium, facilitating its dissolution and absorption, revealing itself as an effective chelating and lubricant agent. It is also effective in the dentinal scrap removal during the periodontal therapy, displaying the collagen and facilitating the tack of the conjunctive tissue treated on the root surface. This composition decalcifies the dentine to a depth of 10 to 30 µm in 5 minutes. Its solubility arrives to 30% and its pH between 7,0 and 8,0 is compatible with living tissues, conferring to the product practically null tissular irritation.

USE INSTRUCTIONS:

I - Endodontics:

1. Dispense 1 drop of **TRISSODIUM E.D.T.A. Liquid** in the entrance of radicular conduct;

2. Wait until the product start acting for 2 to 3 minutes and start instrumentation;

After 5 or 6 minutes the product is going to lose it's effect, because it already reacted with the dentin and was neutralized;

3. Rinse and if necessary repeat the procedure as many time as necessary, placing another quantity of the product;

4. Keep root canal moistened with **TRISSODIUM E.D.T.A. Liquid** during all the operating procedure;

5. After finishing instrumentation, rinse thoroughly with an irrigator solution of your choice. (a.e. Sodium hypochlorite solution, Tergipol or Tergencal).

II - Prosthesis:

1. After prepared the conduct, place few drops of **TRISSODIUM E.D.T.A. Liquid** in the interior of the conduct and leave it acting during 5 minutes;

2. Rinse with an irrigator solution of your choice (a.e. Milton's Liquid, Dakin's Liquid, Chlorinated Soda, Tergencal and Tergipol);

3. Repeat procedure, if necessary;

4. Dry the conduct using absorbing paper and cementing according to the techniques indicated by the chosen product.

SPECIAL CARE:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

TRISSODIUM E.D.T.A. Liquid should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

FOR USE IN DENTISTRY ONLY