COMPOSITION:

CLARIDEX ENDO Powder: Sodium Perborate .

CLARIDEX ENDO Liquid: Hydrogen Peroxide 20% and deionizer water.

INDICATION:

CLARIDEX ENDO is a Dental Bleaching System based on sodium Perborate and hydrogen Peroxide 20% for professional use, indicated for beaching of non-vital teeth.

CLARIDEX ENDO must be administered inside of pulp chamber of the teeth to be teeth to be bleach and should remain for a period of time to oxidize the pigments.

TECHNICAL INFORMATIONS:

The color dimensions (hue, chrome and value) are influenced by presence of dental pulp. When this one ceases to exist, for example, in cases of necrosis of pulp, may occur an alteration in the color and brightness of tooth, which may have a darker tone and shades that may vary between gray green, pardon and blue. The teeth can present alterations of the color due to a number of factors. For successful of bleaching treatment, is important know the of stain tooth. The main causes of changes of color in the crow of pulp less teeth are: pulpal haemorrhage caused by trauma-tism; haemorrhage after pulpal removal, insufficient coronal opening, decomposition of the coronel pulp, medicines and substances used on the canal treatment, permanent and temporary restorative materials and pulp calcification. The teeth darkness process occurs due to the formation of chemically stable structures, responsible for the gradual installation of stains on the teeth crown. The basic process of bleaching involves the oxidation, which is a chemical reaction where organic materials are converted to carbon dioxide and water.

Pigments are composed large quantities of carbon molecules. These are broken and converted in intermediate compounds (minor chains) which are clearer. During the clearing, the carbon chains are transformed in carbonic gas and water, and gradually released along with the oxygen formed by the decomposition of hydrogen peroxide.

CLARIDEX ENDO is a bleaching system based on Sodium Perorate and Hydrogen Peroxide 20%, for professional use. Sodium Perborate is a powder substance that in contact with water decomposes in Sodium Metaborete, Hydrogen Peroxide and oxygen. Used in association with Hydrogen peroxide for clearing of teeth non-vital.

For teeth restoration, after bleaching, is recommended to wait 7 a 15 days, because this interval occur the final colour stabilization and also eliminate of derivatives of oxygen, which can interfere in the polymerization of adhesives systems and composite resins that are used in restorative treatment.

INSTRUCTIONS FOR USE:

- 1.Verify through the clinical examination and radiographic the level of endodontic filing;
- 2.Prophylaxis;
- 3.Registration of current color to compare later;
- 4.Absolute isolation to avoid the contact of the product with the soft adjacent tissues;
- 5.Clear the pulpal chamber;
- 6.From the limit cementum/enamel (verified in radiography), remove 2-3 mm of filling material beyond of the anatomical limit and make a cervical cover about 2mm (recommended to use glass ionomer chemically activated or other sealer of your choice). This sealing is to avoid the permeation of excess oxygen for dentin / filling endodontic interface and periodontium minimizing the chances, of pericementite and external root resorption .
- 7.The make conditioning acid of all pulpal chamber and of the cavosurface angle of coronary opening, wash in abundance and dry. This procedure aims to remove the “smear layer” that is formed during the instrumentation of the cavity and facilitates the action of **CLARIDEX ENDO**; the conditioning of cavosurface facilitates the posterior retention of temporary restoration .
- 8.Mix the **CLARIDEX ENDO** Power and **CLARIDEX ENDO** Liquid until forming a consistent paste, and insert this paste inside of the pulpal chamber.
- 9.Seal the cavity with a provisional restorative (f. ex. BIOPLIC).
- 10.Reevaluate the case after 5 to 7 days.
- 11.If necessary, repeat the application to attain effect, reassessing after 3 to 4 days. Not necessary to repeat the acid conditioning.
- 12.Completed the bleaching treatment, is recommended to apply a paste of calcium hydroxide inside of pulpal chamber for about 7 to 14 days, in order to neutralize the acid pH generated by bleaching material, avoiding the reabsorption cervical and, in the same way, prevent that the present oxygen can interfere in the polymerization of composite resin and impair its union to cleared tooth.
- 13.After the end of the application period of neutralizing paste, realize the restoration of cleared tooth.

Important: In case of modification of the product, contact the company.

Complementary information to the patient:

- 1.Counsel the patient look for immediately the professional care:
- If the temporary restoration unlesh

- If occur intense internal pressure in the pulpal chamber, in this case the temporary restoration must be removed and the biomechanical sealing be reassessed.
- If notice that the dental element is bleaching very quickly. The professional should evaluate the necessity to interrupt treatment early.

2.Avoid the imposition of forces on the tooth being cleared by, the fragility of the tooth during the bleaching treatment.

Precautions to security use:

- 1.During the product handling ,the professional and auxiliary should be using gloves and goggles
- 2.The patient should use goggles and other occasional protections to prevent any accidental contact of the product with the skin, mucous or clothings.
- 3.Realize an efficient isolation of the soft tissues.
- 4.Protect the hands and eyes before opening the bottle of peroxide, because it can be under pressure and/or have casting due the adverse conditions of storage. Casting of small proportions not undertakes to use the product. In case of largest leaks contact the company. Open the lid carefully to inside pressure.
- 5.After use, wash hands well and the materials that come into contact with the product.
- 6.Avoid inhalation of dust had sodium perborate.

SPECIAL CARES:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

CLARIDEX ENDO is contraindicated for people with sensitivity to any components of the product.

Should not be used in pregnant women and breastfeeding.

WARNINGS:

- Avoid the product contact with the eyes.

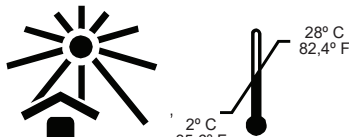
Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

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