- 2. Apply **BIOPLIC** in one block only into the cavity;
- 3. Use an appropriate plugger to mold the desirable shape;
- 4. Light cure for 40 seconds;
- 5. Check the occlusion and wear it when necessary.

REMOVAL:

- 1. Use instrumental to remove **BIOPLIC** from the ends.;
- 2. Proceed permanent treatment;
- 3. Do not reuse the removed BIOPLIC material.

II - As temporary screw sealing on the implant:

APPLICATION:

- 1. Place a layer of **BIOPLIC** on the screw;
- 2. Light cure for 40 seconds;
- 3. Check the occlusion with an articulating paper.

REMOVAL:

- 1. Remove **BIOPLIC** with an explorer;
- 2. Proceed suitable treatment;
- 3. Do not reuse the removed **BIOPLIC** material.

III - As matrix occlusal sculpture in posterior teeth restored with composites:

- 1. Isolate the occlusal surface with Glycerin. Then, apply a thick layer of **BIOPLIC** on tooth occlusal surface adjusting **BIOPLIC** to mold according to the tooth sculpture;
- 2. Light cure for 20 seconds;
- 3. Remove the obtained matrix carefully;
- 4. Proceed restoring treatment using composite following manufacturer's instructions. Fit BIOPLIC occlusal matrix before light cure the last composite layer and softly press it to mold the surface accordingly;
- 5. Remove excess material and keep the matrix still;
- 6. Light cure for 20 seconds;
- 7. Remove the matrix and then light cure the composite according to manufacturer's instructions.

SPECIAL CARES:

After using keep the syringes tightly closed to avoid exposure to the light.

It should not be used when humidity control is impossible.

Dispose of the product in accordance with local legislation, ensuring its complete mischaracterization in order to prevent its reuse and environmental damage.

CONTRAINDICATION:

BIOPLIC should not be used if the patient is known to have sensibility to any of the ingredients listed.

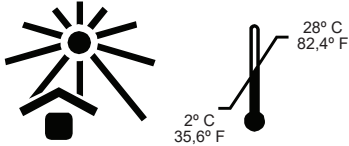
WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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ANVISA nº 10298550053.

COMPOSITION:

Methacrylate groups; organic load; silicon dioxide; catalysts and stabilizer.

INDICATION:

Indicated to seal temporarily cavities in Dentistry, Prosthesis and Endodontics.

In the temporary sealing of screw on the implant.

As a matrix on occlusal sculpture in posterior teeth restored with composites.

TECHNICAL INFORMATION:

BIOPLIC is a resinous material, composed of dimethacrylate groups, inorganic filler, silicon dioxide, catalysts and stabilizer, which after activation by visible light (light-curing device), and hardens to a rubbery consistency.

When used as a temporary restorative material, it seals access to the cavity protecting the exposed dentinal tubules or the entry of the root canal, preventing the passage of fluids, bacteria and toxins.

When used to temporarily seal the screw on implant, its action is to form a mechanical barrier isolating the implant screw from contact with fluids in the oral cavity.

When used as an occlusal sculpture matrix, its action is plastic and modeling, due to the characteristic of the product.

The exposure time of the product to the light of the light-curing device (single exposure) is 40 seconds (as a temporary restorative material) or 20 seconds (when making an occlusal matrix).

It is compatible with composite resin restorations as it does not contain Eugenol.

The recommended time the product remains in the patient as a temporary restorative is up to 30 days. In cases of temporary sealing of the screw on implant, replace it every 30 days.

G.H.F. Desensitizer, when present, is indicated to reduce and prevent dentin hypersensitivity and postoperative sensitivity, making the provisional restoration phase more comfortable for the patient.

USE INSTRUCTIONS:

I - As temporary restorative:

APPLICATION:

- 1. Clean and dry the cavity;