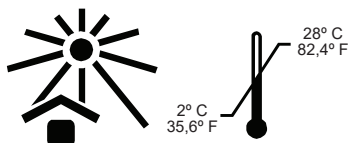
PROVITEMP

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PROVITEMP

Temporary Cement for Veneers, Crowns, Bridges,
Inlays and Onlays

biodinâmica

CONTENTS:

1x2,5g syringe **PROVITEMP Base**
1x2,5g syringe **PROVITEMP Catalyst**
6 application tips.
ANVISA nº 10298550051

COMPOSITION:

PROVITEMP Base: Acrylates and Dimethacrylates Groups; Catalyst; Stabilizer and Thickener..

PROVITEMP Catalyst: Acrylates and Dimethacrylates Groups; Benzoyl peroxide; Stabilizer and Thickener

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TECHNICAL INFORMATION:

PROVITEMP is a temporary cement, Eugenol free (causes non-irritation).

PROVITEMP is transparent shade (does not interfere in the color of cementation), does not suffer erosion in buccal fluids, and has enough resistance to the retention of the restoration and to prevent infiltration. It presents enough hardness to resist the impacts of the chewing. However it is easy removed when necessary .

Its use as a temporary cement is recommended for 15 days only.

USE INSTRUCTIONS:

1. Use the mixing pad and a spatula;
2. Place in the mixing pad equal amounts of the base and catalyst;
Note: After the use, cover the syringes properly.
3. Dry the prepared tooth and the internal surface of the prosthetic piece (crown, inlay, onlay)
4. Mix the base and the catalyst around 10 seconds, until getting a homogeneous product;
5. Apply an uniform layer of this mixture on all the areas that are in contact with the prepared teeth;
6. Accommodate firmly the prosthetic piece in the mouth;
7. Remove the excess of the cement after setting.

NOTE:

Setting time: 1 to 2 minutes.

SPECIAL CARE:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

PROVITEMP should not be used in patients in known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY