1. Remove granulation tissue.
2. Apply **E.D.T.A. Trissodico GEL**, wait 2 to 3 minutes and perform the root scraping.
3. Repeat the operation if necessary.
4. Wash well with saline solution.
5. Follow the procedure, according to the case.

III - For pre-cementation cleaning of posts:

1. After preparing the conduit, insert **E.D.T.A. Trissodico GEL** with the aid of applicator nozzles.
2. Let the product act for 5 minutes.
3. Irrigate with an irrigating solution (eg Dakin's Liquid, Milton's Liquid, Chlorinated Soda, Tergipol and Tergental).
4. Repeat the operation if necessary.
5. Follow the cementation protocol, according to the selected technique and material.

#### **SPECIAL CARE:**

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

#### **CONTRAINDICATION:**

Contraindicated for those allergic to any of the components of the product.

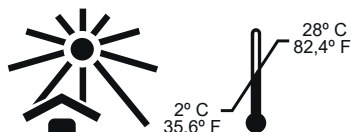
#### **WARNINGS:**

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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ANVISA nº 10298550072

COMPOSITION:

Ethylene diamine tetraacetic Acid; Sodium Hidroxide; Deionized Water and Thickener.

INDICATIONS:

To facilitate endodontic instrumentation;

In the etching of the tooth root in periodontic surgery.

As an auxiliary in the root scaling and to help the cleaning of the conduit for the cementing of pins.

TECHNICAL INFORMATION:

The Trissodium E.D.T.A. is a widely used product in the preparation of root canal walls, previously to the filling. It acts demineralizing dental particles by chelating the ions of calcium and magnesium, facilitating its dissolution and absorption, revealing itself as an effective chelating and lubricant agent. It is also effective in the dentinal scrap removal during the periodontal therapy, displaying the collagen and facilitating the tack of the conjunctive tissue treated on the root surface. This composition decalcifies the dentine to a depth of 10 to 30 µm in 5 minutes. Its exclusive formularization in gel form and its high concentration (24%) results in a bigger effectiveness of the product. The 24% concentration does not promote inconvenience for the dentine or the periapex, but it confers to the product a faster and efficient action. The product takes from 2 to 3 minutes to start making effect and for 5 to 6 minutes it reacts with the dentine and is saturated with calcium ions, losing its effect and neutralizes itself. The pH of the product is approximately 8,5 to 9,0. The viscosity of the gel was especially developed to provide the draining of the product for all the wall of the canal root. The gel is hydro-soluble allowing its complete elimination of the canal root through the cleanness with the indicated irrigator solutions.

USE INSTRUCTIONS :

I - Endodontics:

1. Place a small amount of **E.D.T.A. Trissodium GEL** at the entrance of the root canal at the beginning of instrumentation.

2. Wait 2 to 3 minutes to start instrumentation.

Obs.: after 5 to 6 minutes the product will lose its effect, as it has already reacted with the dentin and has been neutralized.

3. Irrigate and, if necessary, repeat the operation, starting from item 1, as many times as necessary.

4. Keep the canal moistened with **E.D.T.A. Trissodium GEL** throughout the surgery.

5. After instrumentation is completed, irrigate the canal with an irrigating solution (e.g. Dakin's Liquid, Milton's Liquid, Chlorinated Soda, Tergipol and Tergental).

II - Periodontics: