
ÓXIDO DE ZINCO

Material Restaurador Temporário

biodinâmica®

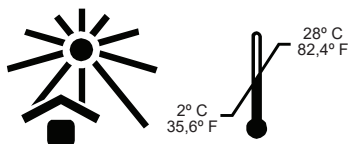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1 pot of 50g
ANVISA nº 10298550062

INDICATION:

It is used as a temporary restorative agent, cavity lining agent, agent for temporary or permanent cementation; agent for root canal filling or as a surgical cement.

COMPOSITION:

Zinc Oxide (99 to 100,5%).

TECHNICAL INFORMATION:

It is a very thin powder, white or rancid-yellow, light, odorless, tasteless, free of rough particles. Its use in dentistry is generally associated with Eugenol, and it is used basically as a temporary material for closing cavities, as well as closing pulp chambers in endodontic procedures. As a provisional material, it maintains its characteristics for a period up to 3 months after the application.

INSTRUCCIONES FOR USE:

1. For temporary restorations: the mixture must be done to obtain a consistent paste that does not stick to the plate or the spatula;
2. For deep cavities with anodyne effect: the paste must be creamy, allowing the contact of Eugenol with the dentin. Over this paste, a new layer, more consistent, must be placed;
3. For cementation and obturation of root canals: the mixture must contain a creamier paste;
4. For surgical cement: the paste must be very consistent, allowing its manipulation with the fingers.

SPECIAL CARES:

Dispose the product according to local legislation, ensuring its complete decharacterization in order to prevent its reuse and environmental damage.

CONTRAINDICATIONS:

Contraindicated for people with reported sensitivity to the product.

WARNINGS:

Do not use products beyond the indicated validity period.

The manufacturer is not responsible for damages caused by incorrect or unforeseen use in the instructions for use.

“KEEP THE PRODUCT OUT OF THE REACH OF CHILDREN”.

EXCLUSIVE USE OF THE DENTAL SURGEON

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