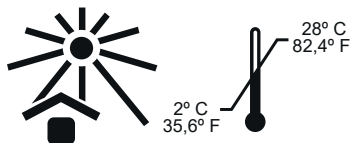
ENDO PTC

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ENDO PTC

Auxiliary material for the instrumentation of the root canal

biodinâmica®

CONTENTS:

1x25g flask of **ENDO PTC**;
or
2x3g syringes of **ENDO PTC**
3 application tips.
ANVISA nº 10298550107.

COMPOSITION:

Carbamide Peroxide, Polysorbate 80, Polyethyleneglycol

INDICATION:

Auxiliary in the chemical-mechanical root canal preparation, to be used alternately with sodium hypochlorite (Dakin Liquid or Milton Liquid)

TECHNICAL INFORMATION:

Each component of the **ENDO PTC** has a function:

The polyethylene glycol is the vehicle and serves a substance transport, becoming its association viable. The formulation vehicle is a grease substance, endowed of a detergent capacity totally water soluble and stable even at ambient temperature. This characteristic of being a grease substance makes possible its lubricant performance during the instrumentation, allowing a soft and affective development, keeping the necessary rhythm. Contributes to diminish the instruments impediment risks at the wall canal, preventing breakings and others accidents.

The polysorbate 80 is a detergent which acts diminishing the superficial tension, allowing that the product spreads in a fast and uniform way at all the canal root surface, favoring the substance penetration in the dental tubules. It has also humectants and emulsifiant characteristics, absorbing the fat and keeping it in suspension, letting a facilitated withdraw;

The urea peroxide acts together with the sodium hypochlorite creating a reaction between them. This reaction promotes a big liberation of oxygen, in a effervescency process which, by mechanical action, eliminates from the root canal the rests of the dentin created by the instruments, organic substances, bacteria and toxic products.

USE INSTRUCTIONS:

1. After the pulptomy and the fixation of the length of the work, irrigate abundantly the pulp chamber with Sodium Hypochlorite (a.e. Dakin Liquid or Milton Liquid);
2. Fulfill the pulp chamber with **ENDO PTC**, select the adequate file (manual or rotatory), proceed the instrumentation in order to remove the dentin dregs;
3. Irrigate abundantly with Sodium Hypochlorite (a.e. Dakin Liquid or Milton Liquid);
4. Apply **ENDO PTC** as much as necessary;
5. In cases of retreatment, the **ENDO PTC** may be used together with the solvent to help the removal the final part of the gutta-percha point, followed by the same canal root prepare procedures.

OBS.: In all cases, **ENDO PTC** must be completely removed before the root canal filling or the long term curative and should be used under an absolute insulation.

SPECIAL CARES:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

ENDO PTC should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY