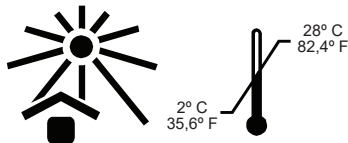
BIOFORTY

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BIOFORTY

Light Curing Surface Sealant for Restoration

biodinâmica®

CONTENTS:

1x5mL flask **BIOFORTY**.
Reg. ANVISA: 10298550098

COMPOSITION:

Bisphenol A glycidylmethacrylate; dimethylmethacrylate groups; silicon dioxide; catalyst; stabilizer and preservative.

INDICATION:

BIOFORTY is used with composites and glass ionomer restorations, on posterior and anterior teeth after finishing and polishing. It is also used to seal the margins of amalgam restorations.

TECHNICAL INFORMATION:

BIOFORTY is an occlusal sealant system for restorations. Light curing material.

It can be used on composites, glass ionomers or amalgam restorations.

Its moderate viscosity allows the flow on the tooth-restoration interface, covering micro leaking regions and improving the lifetime of these restorations.

It can also be applied on pre-existing restorations and must be re-applied at each visit of the patient (each 6 months approximately).

USE INSTRUCTIONS:

1. After restoration is done (finishing and polishing) start etching procedure using 37% phosphoric acid (a.e. Attaque Gel) on all tooth-restoration interface during 30 seconds;
2. Rinse thoroughly all conditioned area for 30 seconds;
3. Apply a thin layer of **BIOFORTY**;
4. Light cure during 20 seconds;
5. Check the occlusion.

NOTE:

In case of pre-existing composite restorations, proceed the prophylaxis with pumice powder and water (this is to remove the protein film and contaminating agents from the surface of the tooth/restoration and then follow the same above instructions.

SPECIAL CARES:

- In case of contact of the product (not light cured) with skin or eyes, irritation can be noticed.
- In case of contact with eyes, immediately rinse thoroughly with water and consult a doctor.
- In case of contact with skin, immediately rinse thoroughly with water and soap.
- In case of incidental ingestion of considerable volume of **BIOFORTY**, DO NOT INDUCE VOMIT and consult a doctor.

SPECIAL RECOMMENDATIONS:

- It is recommended the use of latex gloves, disposable masks and protection glasses when manipulating the product.

-After use, the perfect seal of the bottle must be observed to prevent the material from being exposed to light.

- Do not store **BIOFORTY** near other products containing Eugenol, it interferes at the polymerization.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

BIOFORTY should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY