THEROS

ADESIVO AUTOCONDICIONANTE UNIVERSAL Adesivo Monocomponente para Esmalte e Dentina Fotopolimerizável

biodinâmica®

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1x4 mL e 1x5 mL frasco THEROS

ANVISA nº: 10298559041

COMPOSIÇÃO:

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THEROS

UNIVERSAL SELF-CONDITIONING ADHESIVE Light-cured Single-component Adhesive for Enamel and Dentin

biodinâmica®

PRESENTATION:

1x4 mL and 1x5 mL bottle of THEROS

ANVISA No.: 10298559041

COMPOSITION:

Matrix of methacrylic monomers; phosphate acid monomer (10 MDP), ethyl alcohol, deionized water, initiators and stabilizer.

INDICATION:

1. For direct restorations that require the use of adhesive systems;
2. For indirect restorations, in adhesive cementations;
3. Repairs to porcelain/ceramics and composites.

TECHNICAL INFORMATION:

THEROS is a light-cured, ethanol-based universal adhesive that combines acid, primer and adhesive in a single bottle. It was developed to promote adhesion to dentin and tooth enamel in direct and indirect restorations, with or without prior conditioning. It can be used in 3 different techniques: self-etching on both substrates, total etching (enamel and dentin) and selective enamel etching. In the self-etching adhesive technique, the conditioning of the dental substrate occurs simultaneously with the application of the adhesive, by means of an acid monomer present in the formulation, by which the smear layer is modified and incorporated into the hybrid layer. Self-etching adhesives when used in this technique eliminate the steps of acid etching and washing of the dentin substrate, simplify the adhesive technique, eliminate the risk of collagen fiber collapse and postoperative sensitivity. Also, as an advantage, this type of adhesive can partially demineralize the dentin and allow the residual hydroxyapatite to serve as a receptor for chemical

bonding with the functional monomer of the adhesive and, consequently, improve adhesion, in addition to micromechanical retention. In the total etching technique, the smear layer is removed by phosphoric acid, previously applied to the tooth structure. In selective etching, only the enamel is pre-conditioned and the dentin is self-etched during the application of the adhesive. The viscosity of the adhesive allows for proper penetration and wetting for a homogeneous stable adhesive bond.

INSTRUCTIONS FOR USE:

NOTE: Adhesive procedures should take place in clean cavities, under absolute isolation from the operative field.

Prior preparation:

SELF-CONDITIONING TECHNIQUE: Prepare the cavity. Rinse thoroughly with water. Use absorbent paper or light air blast for 1 to 2 seconds to remove excess water (do not dehydrate the dentin).

TOTAL CONDITIONING TECHNIQUE: Prepare the cavity. Rinse thoroughly with water. Condition the enamel and dentin with a conditioning agent (e.g. Attaque Gel) for 15 seconds. Wash thoroughly. Remove excess water with absorbent paper or light air blast for 1 to 2 seconds, to remove excess water (do not dehydrate).

SELECTIVE ENAMEL ETCHING TECHNIQUE: Prepare the cavity. Rinse thoroughly with water. Condition only the nail polish with a conditioning agent (e.g. Attaque Gel) for 15 seconds. Wash thoroughly. Remove excess water with absorbent paper or light air jet for 1 to 2 seconds to remove excess water.

ADHESIVE APPLICATION:

1. Dispense 1 to 2 drops of **THEROS** into a dappen jar/dispenser pod. Cap the bottle immediately to prevent evaporation of the solvent.
2. Apply two alternating coats of **THEROS**, rub over the preparation for 10 to 15 seconds, each layer. Do not light-cure between layers.
3. Evaporate the solvent with an air blast for at least 10 seconds. The surface should have a shiny, uniform appearance.
4. Light-cure for 20 seconds.
5. Follow the restorative procedure, according to the selected technique and the instructions recommended by the manufacturer of the material.

CONTRAINDICATIONS:

Contraindicated for people with reported sensitivity to any of the components of the product.

PRECAUTIONS:

- Prevent the product from splashing into the eyes. If contact with the eye area occurs, rinse thoroughly with running water. If irritation persists, consult a doctor;
- When using dental adhesives, contamination with saliva will compromise adhesion to the dentin and result in a decrease in the longevity of the restoration. Rubber dam isolation is recommended for all direct restorations or other indications when contamination may occur.
- **THEROS** is a photosensitive material. Use immediately once dispensed onto a glass plate or mixing block. Prolonged exposure to air and light can cause solvent evaporation and thickening of the adhesive.
- To prevent the adhesive from evaporating, keep the bottles properly closed.
- Avoid contact with the skin. Non-polymerized materials such as **THEROS** may cause skin sensitivity in susceptible persons. In case of contact, wash the skin thoroughly with soap and water.
- If materials containing zinc oxide and eugenol have been used in the tooth structure within 2 to 3 weeks, properly clean the tooth surface with an abrasive before using the adhesive.
- The light tip of the light-curing light should be kept as close and vertical as possible to the surface of the resin. If you want to light-cure a large resin surface, it is advisable to divide the area into several sections and light-cure each section separately.
- A low light-curing light intensity will cause poor adhesion.
- When using the light-curing device, follow the manufacturer's guidelines regarding eye protection;
- Do not store in the vicinity of products containing Eugenol, as this fact interferes with polymerization.
- Dispose of the product in accordance with local legislation, ensuring its complete decharacterization in order to prevent its reuse and environmental damage.

WARNINGS:

Do not use products beyond the stated period of validity.

Selective enamel etching is recommended, especially on anterior teeth, to avoid marginal staining.

The manufacturer is not responsible for damage caused by incorrect use or not provided for in the instructions for use.

KEEP THE PRODUCT OUT OF REACH OF CHILDREN.

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