without the cutting extremity. (Initiate for the interproximal space where the ridge generally is deeper and then proceeds the complete return from the tooth in the clockwise one).

3. If necessary, to facilitate the removal, slightly humidify the retraction cord with water.

IDEAL TIME OF PERMANENCE OF THE PRODUCT:

When not using hemostatic solution, the ideal time of permanence of the product for the attainment of the gingival displacement is at least 5 minutes.

When using hemostatic solution, follow the time indicated in the manufacture's instructions.

When moldings, leave the cord located until immediately before impression taking.

PRECAUTIONS FOR A SECURE USE:

This product only should be used on a healthful tissue, free from wounds, bleedings or infections.

When using a hemostatic/retractive solution, follow the solution's manufacturer instructions for the impregnation.

For cervical lesions, the retraction cord should be inserted before the cavity preparation.

When the preparation involves the proximal areas, it might be necessary to use two retraction cords, in order to get a better reproduction of the gingival margin, once is more difficult to displace to retract the interdental gingiva.

After a procedure using the retraction cord, inspect the gingival sulcus to verify if had happened any lesion and to certify that there are no retraction cords fragments on the gingival cervix, avoiding that a strange body causes a local inflammation.

SPECIAL CARES:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

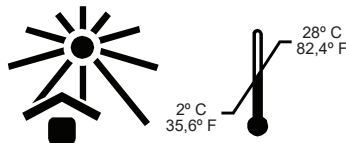
RETRAFLEX should not be used in patients is known to have sensibility to the product.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions. "KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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100% cotton cord.

INDICATION:

RETRAFLEX is indicated for gingival displacement in different dental procedures, such as those that intend to displace the free marginal gingiva, in order to view and manipulate the cervical region of a tooth (preparation, impression, cementing). It also controls the gingival sulcus fluids, optimizing the results of dental impressions, facilitating the access and visibility of all the tissues in which is operating as well as it helps to prevent damages in the gum tissue while preparing the cavities procedures.

TECHNICAL INFORMATION:

RETRAFLEX is non impregnated cotton cord for gingival displacement with mechanic action through its interposition between the free gum edge and dental surface. It facilitates the access and visibility of tissues, besides helping to prevent damages on gingival tissues.

USE INSTRUCTIONS:

1. Cut a piece of **RETRAFLEX** in the desired length or use the individual packing (Uni Dose).

Note: The cord could be absorbed in hemostatic solution to get a better gingival displacement.

2. Place the cord around the neck of the tooth, and introduce in the ridge compressing carefully using a spatula