b) Perform instrumentation, irrigating with physiological saline when saturation occurs;

c) Repeat this procedure as necessary;

d) After instrumentation is complete, irrigate with physiological saline.

II. IN CAVITY ASEPSIS PREPARATION:

a) Clean the cavity with a small amount of CLOREXORAL GEL 2% using a cotton ball;

b) Remove excess CLOREXORAL GEL 2% and dry the cavity with airflow. The use of water in the cavity is optional. Residues of CLOREXORAL GEL 2% do not interfere with material adhesion. uct;

SPECIAL CARES:

Do not ingest.

Do not use as an oral rinse.

Avoid unnecessary contact with the tongue and other parts of the mouth.

CLOREXORAL GEL 2% has an extremely bitter taste and may temporarily alter taste and cause a warming sensation when in contact with the tongue.

Dispose of the product in accordance with local regulations, ensuring complete destruction to prevent reuse and environmental damage.

CONTRAINDICATION:

CLOREXORAL GEL 2% should not be used in patients known to be sensitive to any of the listed ingredients.

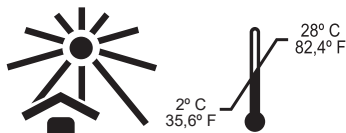
WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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2x3g syringes CLOREXORAL Gel 2%
3 application tips.
ANVISA nº 10298550089.

COMPOSITION:

CLOREXORAL Gel 2%: Chlorhexidine Digluconate (2%), Methylparaben, Hydroxyethylcellulose and Deionized Water.

INDICATION:

CLOREXORAL GEL 2% is used for root canal irrigation, aiding in endodontic instrumentation, and cavity asepsis preparation

TECHNICAL INFORMATION:

Microorganisms and their metabolic products are considered responsible for pulpal and periapical pathologies. Therefore, thorough cleaning of root canal systems is essential for successful endodontic treatment. The principal active ingredient in CLOREXORAL GEL 2% is Chlorhexidine Digluconate. Chlorhexidine was introduced over 30 years ago as a general disinfectant effective against gram-positive and gram-negative bacteria. Chlorhexidine, at low concentration (0.12%), acts as a bacteriostatic agent, while at high concentration (2%), it becomes a fast bactericidal agent.

Chlorhexidine acts through an electrostatic interaction between its positive and negative charges with the bacteria's cellular wall, disrupting it, precipitating the cytoplasm, and causing bacterial death. It exhibits substantivity, with a residual effect lasting approximately 48 hours, and has low toxicity without harming periapical tissue.

USE INSTRUCTIONS:

I. AS AN AUXILIARY IN ENDODONTIC INSTRUMENTATION:
a) Fill the canal completely with the prod