3. After this time, with the aid of an endodontic file or by means of reamers, with slight penetration movements, insert the instrument into the cement and carry out the removal.
4. Drip **CITROL** as needed.
5. After the removal of the filling material, irrigate with a tense active solution (a.e. Tergipol). The purpose of this irrigation is to remove by a tense active way all the product remaining inside the root canal.

SPECIAL CARES:

For this procedure, use only **CITROL**.

The mixture with others products is not indicated and may compromises the final result.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

PRECAUTION FOR A SAFETY USE: In case of an accidental exposure to the dental mucosa, a light irritation may occur. Wash thoroughly with water.

CONSERVATION: **CITROL** is volatile and inflammable. After the use keep the flasks well closed. Conserve the product protected from the heat, flames and oxidant agents.

CONTRAINDICATION:

CITROL should not be used in patients is know to have sensibility to any of the ingredients listed.

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CITROL Liquid: Orange terpene.

INDICATION:

CITROL is used not endodontic treatment as solvent of gutta-percha cones and obturating cement, assisting in the de-occlusion of root canals when it is necessary to carry out there treatment of root canals.

TECHNICAL INFORMATION:

When is necessary to retreat the root canal, the filling material must be removed. Knowing the risks of using only mechanical methods to remove the filling material several techniques are proposed, being necessary, the use of solvents in all of them.

All most all of the root canal filling techniques uses endodontics cement and gutta-percha. Gutta-percha is a material made basically of a vegetal resin and is soluble in chemicals solvents, as the orange terpene. The filling cements that present colophony in their compositions (a.e. N-Rickert and Pulp Fill) are also dissolved by the orange terpene, once the colophony is also a vegetal resin. The essential orange oil is also efficient in cleaning the root canal filled by Zinc Oxide and Eugenol cement . Some solvents are used as an auxiliary in removal of the filling material, preventing against undesirable injuries.

CITROL is an alternative for new endodontics treatment, presenting high capacity as a gutta-percha solvent. It is an essential oil, volatile, extracted from de orange coat and has a pleasant smell.
Its main characteristic is that the product is not toxic for the professional, for the patient and for the environment. Presents a fast action and is innocuous to the adjacent tissue and to the tooth.

USE INSTRUCTIONS:

1. Remove the restoration by mechanical methods until the pulp chamber at the entrance of the root canal.
2. Place one or two drops of **CITROL** at the root conduct entrance and let it works from 5 to 10 minutes.