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INDICATION:

MASTER CEMENT TRY-IN is indicated for the chromatic simulation of the resinous cement **MASTER CEMENT LC** and **MASTER CEMENT DUAL**, for the control of the final adaptation adjust of indirect restorations and for the protection of the ceramic surface, already conditioned in laboratory, during the restoration proof.

COMPOSITION:

MASTER CEMENT TRY-IN: Glycerin; Pyrogenic silica and Pigments.

TECHNICAL INFORMATIONS:

MASTER CEMENT TRY-IN is a proof material for cementation when using the resinous cement **MASTER CEMENT LC** and **MASTER CEMENT DUAL**. The product is based on glycerin and is soluble in water. It acts promoting a simulation of the shade that is going to be obtained through the use of **MASTER CEMENT LC** and **MASTER CEMENT DUAL** at the ceramic or composite indirect restorations, before the definitive cementation process. It's presented and corresponds to the same shades of the resinous cement **MASTER CEMENT LC** and **MASTER CEMENT DUAL**, when completely polymerized.

USE INSTRUCTIONS:

- 1 – Select **MASTER CEMENT TRY-IN** at the corresponding shade to the indirect restoration shade;
 - 2 – Place enough quantity at the internal surface of the restoration placing it correctly, without pressuring;
- OBS.: To avoid the risk of fracture of the restoration, do not press heavily and do not check the occlusion in this stage.
- 3 – Do the necessary adjustments, related to the adaptation and proximal contact and verify the esthetic aspect;
 - 4 – Remove carefully all **MASTER CEMENT TRY-IN** proof material, before accomplish the definitive cementation. Material residues can damage the adhesion between the prosthetic piece and the resinous cement.

SPECIAL CARES:

Eliminate completely all cementation proof material (**MASTER CEMENT TRY IN**) before do the definitive cementation. Residues of this material can damage the adhesion between the restoration and the resinous cement. The material can be removed with water or in ultrasound bath.
Observe that exists a slight difference between the shades of **MASTER CEMENT TRY IN** and the correspondent shades of **MASTER CEMENT LC**, before the photo activation, due to the fact that the coloration is changed after the light curing. The shade of **MASTER CEMENT TRY IN** is correspondent to the shade of **MASTER CEMENT LC**, when completely light cured.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION

MASTER CEMENT TRY-IN is contra-indicated for people with related sensitivity to any components of the product.

WARNINGS:

Do not use material after expiry date.
The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.
“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.
FOR USE IN DENTISTRY ONLY