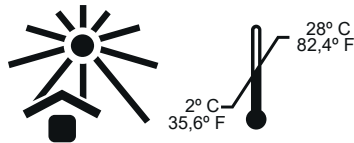
FILLTEMP

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biodinâmica®

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Anvisa nº 10298550113

COMPOSITION:

Zinc oxide, silicone oil, diatomite, calcium sulfate, zinc sulfate, vinyl resin, flavoring.

INDICATION:

Paste for temporary filling in dental procedure. Product ready for direct use into the oral cavity, properly prepared. It has fast chemical hardening that occurs under saliva action and oral temperature, causing its expansion and adequate marginal seal. Setting time varies between 10 to 20 minutes.

TECHNICAL INFORMATION:

FILLTEMP is a temporary obturator used between endodontic treatment sessions and its action is to protect the filled root canal until the final restoration of the tooth is performed. It is a unique material, ready to use and its technique of use is quite simple. Just take it directly into the cavity to be provisionally restored. It presents rapid chemical hardening that occurs under the action of saliva and at oral temperature, causing its expansion and adequate marginal sealing. Setting time varies between 10 to 20 minutes. Its indication is for temporary use, not being recommended for periods longer than 15 days.

USE INSTRUCTIONS:

Product presentation in pots;

Apply **FILLTEMP** direct to the cavity using a spatula and close the flask immediately, preventing the hardening of the material surface. Deep cavities should not be totally filled, allowing removal material easily.

SPECIAL CARE:

- Wait three hours to submit the product to the masticatory force;
- The maximum time the product remain in the patient as provisional restorative is 15 days;
- Dispose the product in accordance with local regulation preventing environment damages.

CONTRAINDICATION:

Contraindicated for patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

FOR USE IN DENTISTRY ONLY