TECHNICAL INFORMATION:

PROVIPLAST is a bis-acrylic resin, self-cured for temporary confections (crowns, bridges, inlays, onlays and facets). Dual-component material (base and catalyst), auto-mixer, with great facility in manipulation and application, low heating in polymerization, low contraction in polymerization, easy polishing, and bio-compatible. Presenting fluorescence, which allows a temporary restoration esthetics and natural.

INSTRUCTIONS OF USE:

- 1 – Before of dental preparation or extraction, hold the impression of the quadrant using the selected molding material;
 - 2 – Dry and lubricate the prepared teeth and the adjacent tissues with vaseline;
 - 3 – To attach the mixing tip on the syringe and expelled a small amount of material before filling the mold;
- OBS: For technical reasons, some little variation may occur while filling the syringes. Thus, remove the syringe cover and proceed the extrusion of the material until exit equally from both plungers, and only then adjust the mixing tip. The material is automatically mixed in the correct ratio when prepare and can be directly applied.
- 4 – Inject **PROVIPLAST** in the deep portion of impression and after in the gingival areas, keeping the applicator tip immersed in the material to avoid bubbles;
 - 5 – Insert the impression, filled with **PROVIPLAST**, within 1 minute, over the prepared area and keep in the local performing pressure for approximately 2 minutes after the start the mixture;
 - Note – The use of lubricant helps in removing.
 - 6 – Remove the **PROVIPLAST** during the elastic phase. The reaction of prey should be intra-orally monitored (f.ex. with a probe or curette) by excess material in the cavity, because the temperature in the mouth has a significant influence in the prey time, and the provisional removal is only possible during the elastic phase.
 - 7- After removing the temporary restoration of the impression or the prepared element (up to 2 minutes), remove the excess material with a cutting or rotating instrument. Wait 5 minutes from the beginning of the process and do the finish and polish of the temporary restoration. Use ethyl alcohol to remove the thin layer of inhibition, oily, which is caused by oxygen on the surface, so that the finishing and polishing becomes easier;
 - 8 – Perform the provisional restoration cementation with a temporary cement (f.ex. PROVITEMP);
 - Note - Cements based on eugenol can inhibit the polymerization of definitive cementation.
 - 9 – Check the occlusion.
 - 10 – In cases of need to repair the provisional, are following the recommended procedures:

Air bubbles – fill with **PROVIPLAST**.

Repair of a new provisional – Connect the fracture point with **PROVIPLAST**.

Repair of a former provisional – Scratch the surface of fracture points, with the help of a drill. Apply an adhesive agent in accordance with the manufacturer's instructions, to dispense a quantity of **PROVIPLAST** on surfaces, hold for 2 minutes. Wait 5 minutes before remove the excess with rotary instruments.

Alternatively, the **PROVIPLAST** provisional can be repaired with compost resin.

SPECIAL CARES:

Avoid direct contact of **PROVIPLAST** with the skin.

In case of the skin contact, wash thoroughly with soap and water.

In case of irritations or allergic reactions, stop the use immediately.

In case of contact with the eyes, wash thoroughly with soap and if persistent irritation, seek a doctor.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRA-INDICATION:

PROVIPLAST should not be used in patients in known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

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