
EUCALIPTOL

Solvente de Gutta-Percha

biodinâmica

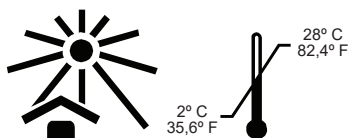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Eucalyptol U.S.P.

INDICATION:

As gutta-percha point solvent, assisting in the root canal restoration removal.

TECHNICAL INFORMATION:

EUCALIPTOL is obtained from essential oil of Eucalyptus taken from several Eucalyptus breeds that present big quantity of 1,8 - Cineol. It is a clean liquid, colorless or pale yellow color, camphor like odor, cooling taste and spicy. Almost insoluble in water, but soluble with alcohol, Chloroform, Carbon Sulfate, Glacial Acetic acid, vegetal oils and essences.

USE INSTRUCTIONS:

1. Place 1 drop of **EUCALIPTOL** in the entrance of radicular conduct and wait for 5 minutes until the product start solving gutta-percha point;
2. Remove gutta-percha point using an endodontic file;
3. In case gutta-percha point remains dry and brittle place **EUCALIPTOL** again;
4. Repeat procedure until it is necessary.

NOTE:

For better results use pure **EUCALIPTOL**. The mixture with other products will make the procedure very slow.

SPECIAL CARES:

Keep the flask well closed, in order to avoid the volatilization of the product.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

EUCALIPTOL should not be used in patients is known to have sensibility to the product.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.
“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY