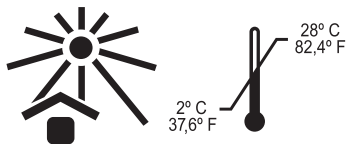
# EVIPLAC PASTILHAS

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# TABLETS EVIPLAC

Disclosing Tablets for Bacterial Plaque

bioDinámica

## CONTENTS:

**TABLETS EVIPLAC** (flask with 60 units).

or

**TABLETS EVIPLAC** (flask with 120 units).

or

**TABLETS EVIPLAC Blister** (10 envelopes with 6 tablets each = 60 units or 20 envelopes with 6 tablets each = 120 units).

ANVISA nº 10298550090.

## COMPOSITION:

Basic Fucsin (2%), Saccharin Sodium, Lactose, Sodium Cyclamate and Excipients.

## INDICATION:

**TABLETS EVIPLAC** is indicated to disclose bacterial plaque.

## TECHNICAL INFORMATION:

The formulation of **EVIPLAC** is based on basic fucsin guarantees to the product a higher performance for disclosing the bacterial plaque, on apparently clean teeth, staining them with a color that contrast to the teeth. Such device not only disclose the plaque but helps in the cleansing instruction as well.

## USE INSTRUCTIONS:

1. Place one tablet in mouth and go rubbing to the teeth, using the tongue, until the complete dissolution;
2. After, rinse with water to remove excess.

## NOTE:

All the areas stained indicate bacterial plaque presence that must be completely removed by toothbrush and dental floss.

Can be daily used and preferably before the last tooth brushing.

## SPECIAL CARES:

The main use of the product is to color the bacterial plaque of a contrasting color with teeth and tissue;

It must be used only as an auxiliar in the disclosing of the bacterial plaque;

Disclosed plaque must be removed when brushing it;

Protect clothes away from the contact with the product once product's stain is difficult to be removed.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

## CONTRAINICATION:

**TABLETS EVIPLAC** should not be used in patients is known to have sensibility to any of the ingredients listed.

## WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

\*KEEP MATERIAL OUT OF THE REACH OF CHILDREN\*.

FOR USE IN DENTISTRY ONLY