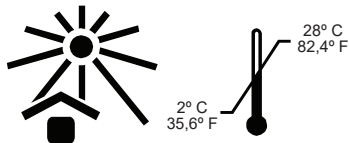
HEMOSTANK

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biodinâmica®

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ANVISA nº 10298550099

COMPOSITION:

Aluminium Chloride 25%, Deionized Water and Methylparaben.

INDICATION:

HEMOSTANK is indicated as a helper in cases where is needed a precision in the border's gum molding, during cavities prepare and it is used together with the retraction cord.

TECHNICAL INFORMATION:

HEMOSTANK is a hemostatic for topical use, indicated to promote hemostasis during the gum removal before the realizations of moldings. Contains Aluminium Chloride in it's composition that acts mainly as astringent. It is considered safe and efficient. The tissue after moved away come back in it's normal condition. Does not interfere in the molding materials setting and can be used by cardiac patients due the absence of epinephrine.

USE INSTRUCTIONS:

As a helper in molding of cavity preparations:

Humidify the retraction cord with the **HEMOSTANK** solution and then proceed in accordance to the indicated technique.

SPECIAL CARES:

Use only the necessary quantity of drops and after getting the desired result, rinse abundantly the local to remove the product.

The product is irritating for the eyes and for the mucosa.

The inadequate use of the product can take the irritation or tissue destruction. The retractor cord impregnated with **HEMOSTANK** do not should be forced to beyond of apical extension of cleft gum, or involve the strangled gum tissue against the dental surface.

Do not use it above a retractor cord already impregnated.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

HEMOSTANK should not be used in patients who are known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY