
INTERIM Líquido

Material Restaurador Temporário

biodinâmica®

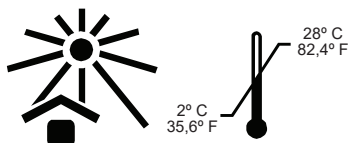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ANVISA n° 10298550041.

INDICATION:

INTERIM Indicated for lining cavities and long-term provisional restorations.

COMPOSITION:

Eugenol and acetic acid glacial (0,5%).

TECHNICAL INFORMATION:

Setting time is 3 to 8 minutes. It is one of the least irritating cements used in Dentistry, due to its pH 7.0 - 8.0, even when the product is inserted into the cavity. In addition, it also has some antiseptic and sedative properties due to the eugenol in its composition.

USE INSTRUCTIONS:

1. Dispense the measure of powder and 1 drop of liquid onto a glass plate, divide the powder into 3 equal parts;
2. Mix 1/3 of the powder with the liquid and spatula for 20 seconds;
3. Add another 1/3 of the powder and spatulate for 20 seconds;
4. Finally add the remaining 1/3 of the powder and mix for another 20 seconds until obtaining a mass with little adherence to the glass plate;
5. Use a Hollemback presser to place appropriate portions of the mass into the cavity, completing the placement with a presser or burnisher as the cement sets.

Excesses can be eliminated with a Hollemback sculptor or with low speed drills.

SPECIAL CARE:

It should not be used with resinous materials as it interferes with their polymerization properties.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

INTERIM should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.
“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY