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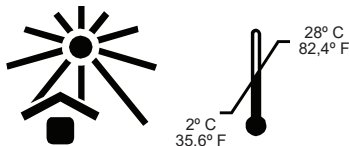
COMPOSITION:

PARAMONOCHELOPHENOL: Chlorophenol; 96º Ethyl Alcohol and Deionized Water.
CAMPFORATED PARAMONOCHELOPHENOL: Chlorophenol; Camphor; 96º Ethyl Alcohol and Deionized Water.
PARAMONOCHELOPHENOL WITH FURACIN: Chlorophenol, 96º Ethyl Alcohol, Deionized Water and Solution of Nitrofurazon 0,2%

INDICATION:

It is indicated as an intracanal curative.
It has non-specific bactericide action destroying a broad spectrum of microorganisms.

TECHNICAL INFORMATION:

As intracanal curative, the use in presented concentration, has demonstrated low tissular reactions and shows no significant effects on the synthesis of proteins, presenting antibacterial short duration effect; Minimum quantity of this product is quite effective to act in whole canal; The presence of Nitrofurazon (**PARAMONOCHELOPHENOL WITH FURACIN**) assures to the product antiseptic and anti-infectious properties.

USE INSTRUCTIONS:

1. After root canal is instrumented and dried, soak a cotton ball in the product and dry it with a small cotton roll;
2. The dry cotton ball with the product is then placed in the pulp cavity floor;
3. Close hermetically with appropriate cement;
4. After 24 - 48hs, remove the curative and fill the root canal.

NOTE:

The product is volatile and has low surface tension. It will fast spread by root canal in the recommended period.

Minimum quantity of this product applied by cotton ball technique is quite effective for the bacterial control.

SPECIAL CARE:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

PARAMONOCHELOPHENOL should not be used in patients is known to have sensibility to any of the ingredients listed.

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Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY