
EVICÁRIE

Evidenciador da Dentina Cariada

biodinâmica

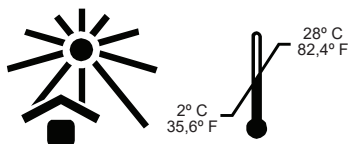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ANVISA nº 10298550088

COMPOSITION:

EVICARIE Liquid: Red Colouring Acid S52, Propilenoglicol, Preservative and Deionized Water.

EVICARIE Gel: Red Colouring Acid S52, Propilenoglicol, Thickener, Preservative and Deionized Water

INDICATION:

EVICARIE is indicated to identify dentin caries

TECHNICAL INFORMATION:

The caries indicators as **EVICARIE** are materials used to assist the professional in the differentiation between the affected dentin and the caries dentin in restoring treatments. The affected dentin is not contaminated with bacteria and even being a more soft tissue than the health dentin, should not be removed while the infected dentin must be removed before cavity restoration.

The **EVICARIE'S** action is noticed through the red colouring acid S52 present in its formulation and the red coloured places indicate the decalcified and infected dentin.

EVICARIE clinically clear differentiates the infected dentin layer that should be removed and do not pigment the others layers or normal dentin.

USE INSTRUCTIONS:

1. Complete isolate the tooth;
2. After washing and drying the caries tooth, apply a drop of **EVICARIE** in the affected area;
Note: To prevent the crossed contamination or infection, do not apply directly the product from the flask or syringe in the carious cavity. (Use a disposable application tip).
3. Wait 10 seconds and wash with water;
4. Remove the external layer of the caries dentin that is red coloured;
5. Repeat this operation as many times as necessary until there is no more red coloured area, assuring the complete removal of the caries dentin;
6. After the removal of the red coloured tissue, the cavity is ready to be restored.

SPECIAL CARES:

To prevent the crossed contamination or infection, do not apply directly the product from the flask or syringe in the carious cavity. (Use a disposable application tip).

Avoid direct contact with the eyes, skin or clothes.

In case of contact with the eyes: rinse thoroughly with water and look for medical assistance.

In case of contact with skin and clothes: wash with soap and water immediately.

Only apply the product in the tooth that is being treated.

Be careful do not put the product in contact with others existing restorations, especially in the marginal areas.

The colourings are difficult to be removed from these areas.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

EVICARIE should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY