
MODELLARE RESIN

Resina Modeladora Fotopolimerizável

biodinâmica®

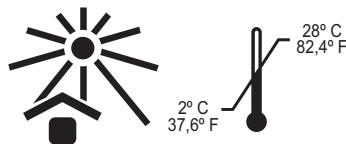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Reg. ANVISA: 10298550188

COMPOSITION:

Matrix of methacrylic monomers, inorganic fillers, initiators.

INDICATION:

Indicated as a humidifying agent and auxiliary for sculpture in composite resin.

TECHNICAL INFORMATION:

MODELLARE RESIN is a low viscosity micro-particulate resin used to facilitate the insertion and carving of composite resin. Its humectant characteristic that improves the sculptability of the composite resin and prevents it from adhering to the instruments. For presenting load and being Solvent-free, the product does not significantly alter the mechanical characteristics of the resins compounds to which it is applied. In addition, it creates an oxygen-inhibiting layer necessary for the application of incremental layers and for resin-resin bonding; for being photopolymerizable, undergoes the polymerization process along with the restorative material.

INSTRUCTIONS FOR USE:

1. Remove the cap, adapt the applicator tip to the syringe. Dispense one or two drops onto a pad mixer or glass plate.

2. Insert the initial increment of composite resin into the preparation, wet the spatula in the **MODELLARE RESIN**. With the **MODELLARE RESIN** on the instrument or brush, sculpt your restoration, leave the resin in the desired shape.

3. Light cure. Although **MODELLARE RESIN** requires light curing for 10 (ten) seconds, follow the curing time recommendations established by the resin manufacturer composite to ensure complete cure of the composite increment. Additional increments of composite resin will adhere to the polymerized **MODELLARE RESIN**.

SPECIAL CARES:

- If the product accidentally comes into contact with the eye, rinse thoroughly with water and seek medical attention immediately. In case of contact with other fabrics, wash immediately with plenty of water for several minutes.

- Cross contamination: the product may contain items designed for single use. To eliminate used or contaminated single-use components and accessories. Do not clean, disinfect nor reuse.

- Non-polymerised resins may cause sensitization in susceptible persons. avoid the skin exposure. In case of contact with the skin, wash thoroughly with soap and water.

- Test flow of syringe and delivery tip materials prior to intraoral application.

- After use, it is recommended to perform a slight traction on the plunger, in order to reduce the pressure inside the syringe and avoid leakage of the product.

- Dispose of the product in accordance with local legislation, ensuring its complete mischaracterization in order to prevent its reuse and environmental damage.

CONTRAINDICATION:

MODELLARE RESIN is contraindicated for people with reported sensitivity to any of the product's components.

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY