In case of contact with the eyes, wash immediately with water and seek medical advice.

SPECIAL RECOMMENDATIONS:

Shake the powder and liquid before using;
Liquid flask must be carefully manipulated to avoid any air gaps;
Use a rigid manipulating spatula, the plate area should be as small as possible;
Material surface must be always shining after spatulation;
Insertion can be done using a spatula, application tips or insertion syringes;
Condesation is made with condensers, burnishers or small ball of polyethylene foam.
Strengthen the material against the walls for bubbles removal and better adaptation.
Surface protection must be realized owing to property that has this material to lose water at the beginning and incorporate after some time, what would cause loss of mechanic propriety.
Right after the use of manipulation instruments, rinse with cold water to avoid solidification and difficult removal of material residuals;
Do not use liquid and powder from different ionomers flasks;
After finish any of **BIOGLASS F** components both flaks should be disposed;
Avoid direct contact with Eugenol based products once it interferes in some glass ionomer's working time.
The use of cooled plate increases working time.
Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

STORAGE:

Do not keep in refrigerator;
Keep the liquid and powder flasks tightly closed;
After use close products immediately;
Do not expose product to intensive light;
In case liquid becomes more viscous immerse it on water at 75°C temperature during 10 to 15 minutes to be reused.

CONTRAINDICATION:

BIOGLASS F should not be used in people is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.
"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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biodinâmica

CONTENTS:

1x10g flask **BIOGLASS F POWDER**
1x13mL flask **BIOGLASS F LIQUID**
1 measuring spoon.

ANVISA nº 10298550064.

COMPOSITION:

BIOGLASS F POWDER: Calcium, Barium and Aluminum Fluorsilicate, Lyophilized Polyacrylic Acid, Barium Sulfate and pigments.

BIOGLASS F LIQUID:Tartaric Acid and Deionized Water.

INDICATION:

BIOGLASS F is indicated for reconstruction of cores where there is still a remaining coronary structure (50%);

For filling of small retention or failures in prepares;

For recovering of hardness of the cusps without dentin support;

In the lining of cavities including light cured resins (sandwich technique);

TECHNICAL INFORMATION:

Glass ionomer is a very important material in the current and modern Odontology, mainly because its adhesion property and fluorine release. Such properties guarantee an excellent cavity sealing and decay prevention. Easy to manipulate. Excellent aesthetics and mechanical properties. Used in reconstruction of prosthetic pieces only in elements with more than 50% of dental structure. Fluor release guarantee an anticariogenic action. Continuous recharged of Fluor considering its bucal absorption. It allows both the remineralization of adjacent tissues and the recovery of rigidity of cusps without dentin support. Chemical adhesion to dental structures through its link with Calcium ions. Higher adhesion to the enamel than the dentin. Similar thermal expansion and contraction to dentin, having no post-operating sensitivity. High adhesivity material having no need of additional retentions. Biocompatible and hydrophilic material. It is the most compatibility material to our dental structure. In spite of initial cement acidity, Calcium Hydroxide based cement is recommended for deep cavities. Fluor contributes to reduce material pH. Radiopaque material and the high adhesivity to the dental structure avoid marginal leakage.

USE INSTRUCTIONS:

PREPARATION OF CAVITY:

1. Prepare the cavity;
2. Clean cavity with water and dry, but not dehydrate;
3. In case of exposed pulp,recommended to apply Calcium Hydroxide cement (a.e. **BIOCAL**);
4. Using a brush apply a dentin conditioner (**BIOGLASS F LIQUID**) during 15 seconds. Wash with water and dry, but not dehydrate.

MIXING, WORKING AND SETTING TIME: (23°C and 50% UR):

Mixing time: 20 to 30 seconds/ Working time: 1 min. to 1 min. 30 sec / Setting time: 5 to 6 minutes.

MIXING RATIO:

- Powder / liquid ratio: 1 (one) measure of powder to 1 (one) drop of liquid
- Ideal ratio: (m/m): 2:1

SHAKE LIQUID AND POWDER FLASK BEFORE USE.

For a precise dosage of the liquid turn the flask vertically down. Release the air from inside the flask. After use, close it tightly both flasks to avoid humidity contact.

SANDWICH TECHNIQUE: for intermediary restorations.

1. Isolate area;
2. Proceed the prepare of the cavity;
3. Wash and dry – no dehydrate;
4. Manipulate the cement;
5. Place cement into the cavity using proper instrument. Avoid air gaps;
6. Await light curing before proceed treatment with final restorative material.

SPECIAL CARES:

Both Liquid and Cement when touching the skin and mucosa are corrosive.

KEEP LIQUID DISTANT FROM THE EYES AREA AND AVOID LONG CONTACT WITH SKIN AND ORAL TISSUE.

In case of contact with oral tissue and skin, remove it immediately with cotton imbibed in alcohol. Rinse with water.