TECHNICAL INFORMATION:

The bonding agent SILANO acts as a connector between the indirect restoration and the composite by means of a stable chemical bond, providing increased bond strength in the indirect/composite restoration interface.

USE INSTRUCTIONS:

FOR CEMENTATION OF INDIRECT RESTORATIONS WITH RESINOUS CEMENTS:

- 1. Prepare the tooth surface according to the technique recommended by the agent cement;
- 2. Apply porcelain etchant (hydrofluoric acid, e.g. Ataque F) to the inner surface of the restoration for the time recommended by the manufacturer. Wash and dry carefully;
- 3. Apply SILANO BONDING AGENT using a brush on the conditioned surface, wait 1 minute;
- 4. Cement the restoration according to the recommendations for the resin cement to be used (ex Master Cement Dual).

FOR COMPOSITE RESIN REPAIRS:

- 1. Prophylaxis of the surface to be repaired with pumice stone and water, dry and isolate;
- 2. Remove friable edges and roughen the surface with diamond burs;
- 3. Etch the surface with porcelain conditioner (hydrofluoric acid, e.x. Ataque F) at least time recommended by the manufacturer;
- 4. Wash thoroughly and dry completely;
- 5. Apply the SILANO JOINING AGENT using a brush over the entire surface conditioned, wait 1 minute;
- 6. Apply the selected conventional adhesive system (ex Master Bond) and light cure as required. manufacturer's instructions;
- 7. Repair the fractured area with a composite (eg Master Fill).

COLLAGE OF FRAGMENTS:

- 1. Perform prophylaxis with pumice stone and water on the surface to be repaired, dry and isolate;
- 2. Roughen the surface to be repaired and the fragment with diamond burs;
- 3. Condition the surface of the fragment and the flaw with porcelain conditioner (acid hydrofluoric, e.g. Attack F) for the time recommended by the manufacturer;
- 4. Wash thoroughly and dry thoroughly;
- 5. Apply the SILANO JOINING AGENT using a brush over the entire surface conditioned, wait 1 minute;
- 6. Cement the fragment according to the cement manufacturer's instructions;
- 7. If there is a lack of material in the fragment (ceramic, porcelain, resin), rebuild the

fractured area with composite (ex Master Fill).
SPECIAL CARES:
- Every procedure must be done preferably with absolute isolation;
- Extensive repairs with composites should be performed in layers to reduce the effect of polymerization contraction.
- Dispose of the product in accordance with local legislation, ensuring its complete mischaracterization in order to prevent its reuse and environmental damage.

CONTRAINDICATION:

Contraindicated for people with reported sensitivity to any of the product's components.

WARNINGS:

Do not use material after expiry date.
The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.
"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".
FOR USE IN DENTISTRY ONLY

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