SPECIAL CARES:

It provokes irritation in case of skin contact. Use adequate clothes for the procedure.
It provokes irritation in the oral mucosa. In case of contact, wash immediately with water.
It provokes irritation in case of eyes contact. In case of contact, wash immediately with plenty of water and seek for medical advice if necessary.
In case of ingestion do not force the vomit. Take big quantities of water or milk.
Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

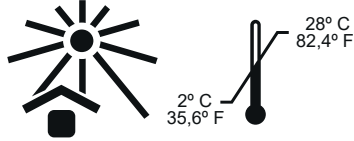
HYDROPAST should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.
“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

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Temporary Canal Root Filling Material

biodinâmica®

CONTENTS:
1x2,5g syringe **HYDROPAST** or **HYDROPAST** + I
+ accessories.
Reg. ANVISA: 10298550081/ 10298550093.
COMPOSITION:
HYDROPAST: Calcium Hydroxide (38%); Barium Sulfate and Vehicle.
HYDROPAST + I: Calcium Hydroxide (38%); Iodoform (26%) and Vehicle.
INDICATION:
HYDROPAST is indicated as a temporary canal root filling material; In exudates control; In periapical injuries; As a inductor in the apexification process; In the root and perforations reabsorption treatment; In cases of exposed pulp, pulp lining and polpotomy; As an auxiliary in the “plug apical” permanent filling technique; In deciduous teeth pulpectomies.
TECHNICAL INFORMATION:
HYDROPAST presents in its composition Calcium Hydroxide, an important element to induce the reparative dentin formation and used as intracanal material in endodontics procedures. The calcium hydroxide provides the product with an antibacterial action, depriving residuals microorganisms of their nutritive supply preventing exudate penetration in the canal root system interior. Its pH is 12,4, highly alkaline, may biased toward bacterial death. Old calcium hydroxide dressings have demonstrated lots of periapicals altering reversions. This component also has an excellent biocompatibility, helping the tissue reparation by tissue enzymatic activation, generating a tissue mineralization. It leads to reparative dentin formation. It allows a calcification increase of the preexistent dentin layer. Offers better protecting conditions to pulp when submitted to injuries. The presence of iodoform in **HYDROPAST** + I gives to the product an anti-infectious property, with prolonged action and intense radiopacity.

USE INSTRUCTIONS:
Use **HYDROPAST** after the instrumentation and drying the root canal.
I. AS A TEMPORARY MATERIAL FOR ROOT CANAL FILLING:
1) Insert **HYDROPAST** in the third apical of the canal length;
2) Press the embolus until the paste gets the apical foramen and then slowly move away the syringe until the canal is completed filled;
3) Remove the paste excess with a sterile cotton;
4) Once the canal is filled with **HYDROPAST**, complete the treatment with a temporary restoration in order to seal the access.
II. AS AN AUXILIAR FOR THE PERMANENT FILLING IN THE “PLUG” APICAL TECHNIQUE:
(Together with gutta-percha and endodontic cement)
1) Select the principal cone;
2) To assure the apical sealing, add a small quantity of **HYDROPAST** in the periapical portion of the canal;
3) Complete the filling with a regular side condensing technique;
4) Do the permanent filling of the access.
III. PULPECTOMY IN DECIDUOUS TEETH:
1) Insert **HYDROPAST** in the canal, from the apex to the coronary access;
BE CAREFUL TO AVOID PASTE IN EXCESS AFTER THE APEX
2) Complete the treatment with a restoration to seal the access.
3) **HYDROPAST** will be reabsorbed during the dental root physiological resorption.