
N-RICKERT

Cimento Endodôntico

biodinâmica

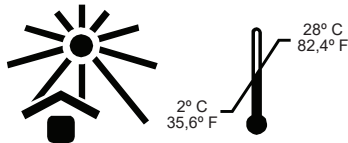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CONTENTS:

Kit contents:

1x10g flask **N-RICKERT Powder**;
1x10mL flask **N-RICKERT Liquid**.

or

1x10g flask **N-RICKERT Powder**;

or

1x10mL flask **N-RICKERT Liquid**.

Reg. ANVISA: 10298550083.

COMPOSITION:

N-RICKERT Powder: Zinc oxide (48%), Staybelite resin (hydrogenated) and Silver precipitate.

N-RICKERT Liquid: Eugenol (70%), Canadian Balsam and Thymol.

INDICATION:

N-RICKERT is an endodontic cement used for filling root canals.

TECHNICAL INFORMATION:

N-RICKERT is an endodontic cement, used as intra canal filling material.

The liquid and the powder provide one viscose and consistent mixture, which is used as gutta-percha point.

The setting of this cement is because of the reaction between the Zinc Oxide and Eugenol, which in the presence of water produces the zinc eugenolate. The resin improves the characteristics of the mixture and delay the setting time. The radiopacity is optimized by the silver.

The formulation of the **N-RICKERT** is similar to the one developed by **RICKERT**, that was one of the first to be used and fulfill the requests of a good endodontic cement: has good adhesion; good penetration; high radiopacity (silver); easy to remove (with alcohol 70 GL); and antibacterial action (eugenol, thymol and silver).

USE INSTRUCTIONS:

1. Place a standard portion of liquid and powder on a glass plate.
2. Mix products until obtain an homogeneous mixing. Use a spatula to obtain a string from the mass. Such string must resist when the spatula is away from the mass and product is ready to be applied.

SPECIAL CARES:

After filling, the pulp chamber must be cleaned with alcohol in order to remove residual of cement. In anterior teeth remove part of the filling just to avoid touching cervix tooth.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

N-RICKERT should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY