- 3. For the application of the **ORTHOPROTECT** do not etch or use bondings or primers;
 - 4. Do not reuse the application tips to avoid cross-contamination;
 - 5. Wear gloves and safety goggles during handling of the product. In case of accidental contact with eyes or other oral tissues wash with abundant water;
 - 6. Do not ingest or aspire the product. If this accidentally occurs, seek for medical attention immediately;
 - 7. Close the syringe with its own cap, after use;
- For your own safety, keep the product in its original packaging.

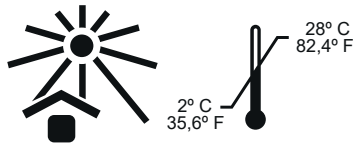
CONTRAINDICATION:
ORTHOPROTECT should not be used if the patient is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.
“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

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ORTHOPROTECT

Protective Coatings for Orthodontic Devices

PRESENTATION:

1X2g ORTHOPROTECT syringe.
03 application tips.
ANVISA nº 10298559029.

COMPOSITION:

Methacrylate groups, aluminum and barium borosilicate glass, silicon dioxide, catalysts, stabilizer.

INDICATION:

ORTHOPROTECT is indicated as a partial component coating of fixed devices used in orthodontic treatments (such as brackets and wires), which have high-pitched corners or sharpened edges that may cause injuries the oral soft tissues.

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ORTHOPROTECT is a light cured composite used to protect the oral soft tissues that may get in touch brackets (metallic, ceramic or plastic) and/or wires, used in orthodontic treatments. When applied at the surface of the brackets **ORTHOPROTECT** generates a physical barrier avoiding the direct contact of the bracket with the adjacent soft tissue preventing the beginning or progression of damage to the tissues. As it is a translucent product, does not interfere at the aesthetics. The low viscosity of the product makes it easy to apply and increases its filling power. Due to its mechanical characteristics, the product can be removed from the surface of the bracket very easily, when necessary.

INSTRUCTIONS FOR USE:

1. Use with orthodontic components:

APPLICATION:

1. Check with the patient if there are areas where the fixed device is uncomfortable;
 2. Make a relative isolation and dry the piece that will receive the **ORTHOPROTECT**;
 3. Remove the syringe's cap place the application tip. Apply the product in the areas previously identified.
- ATTENTION:** Do not apply the product in areas that may interfere in functionality of parts or that undermine the access for the future maintenance;
4. After the application of the product, if necessary, the finishing can be done using disposable applicators (a.e. Master Brush Reg ANVISA nº 10298550122);
 5. Light cure for 30 seconds
 6. If there is occlusal interference due to possible excesses of **ORTHOPROTECT**, proceed with wear using finishing burs.

Note: Check with the patient if the protection is effective, if not, restart the procedure.

REMOVAL:

1. Relieve the retention areas using instruments (a.e. probes or spatulas) or a fine grain bur (a.e. Diamantec Diamond Burs Reg. ANVISA nº 10298550077).

NOTE:

1. Avoid fluid contamination during the application of the product;
2. Avoid the application of the product in areas that prevent the correct manipulation of the orthodontic pieces applied or cause occlusal interference;

SPECIAL CARES:

1. The area must be dry and under relative isolation for the appliation of the **ORTHOPROTECT**;
2. The light curing time for the product depends on the power of light emitted by the equipment. It is recommended that the power density of the light curing equipment is above 400 mW/cm2. We also recommend that the patient and the dentist were googles during the blue light emission by the curing light unit;