PROVIPLAST GLAZE

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PROVIPLAST GLAZE

Light Curing Bisacrylic Resin Sealant and Glazer

biodinâmica

CONTENTS:

1 x 5 mL flask **PROVIPLAST GLAZE**

ANVISA n° 10298550112

INDICATION:

PROVIPLAST GLAZE is indicated for sealing and glazing of temporary crown and bridge surfaces in bisacrylic resins and composite restorations.

COMPOSITION:

Matrix of methacrylic monomers, initiators, stabilizers and thickeners.

TECHNICAL INFORMATION:

PROVIPLAST GLAZE is a one-component material specially developed for surface treatment of bisacrylic resins on temporary crowns and bridges. Its application results in a smooth, shiny, ultra dry and stain resistant finish. The curing time is 20 seconds and a curing light with a minimum power of 400 mW should be used.

USE INSTRUCTIONS:

Surface Sealing:

1. Apply a thin layer of product directly to the surface of the provisional material or composite;
2. For each applied layer, light cure for 20 seconds.

Note: **PROVIPLAST GLAZE** can also be used on pre-existing restorations. For best adhesion, the surface should be roughened, cleaned and dried prior to application of the product.

SPECIAL CARES:

- Contact of non-polymerized product with skin and eyes may cause irritation;
- In contact with eyes, wash with running water and seek medical advice;
- In contact with skin, wash the area with running water and mild soap;
- In case of accidental ingestion of large quantities of the product, DO NOT INDUCE VOMITING and seek medical advice;
- The use of gloves, disposable masks and goggles is recommended to handle the product;
- After use the perfect sealing of the bottle should be observed to prevent the material being exposed to light;
- Do not store in the vicinity of products containing eugenol, as this interferes with polymerization;
- Dispose of the product in accordance with local legislation, ensuring its complete decharacterization in order to prevent its reuse and environmental damage.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

PROVIPLAST GLAZE is contraindicated for people with reported sensitivity to any of the product's components.

WARNINGS:

Do not use material after expiry date;
 Keep material out of the reach of children;
FOR USE IN DENTISTRY ONLY;

The manufacturer is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

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