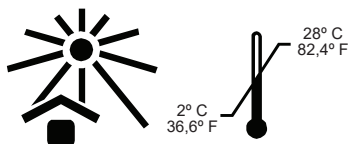
EUGENOL

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biodinâmica

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Eugenol U.S.P(99 to100%)

INDICATION:

EUGENOL is used in reaction with Zinc Oxide powder, in various dental operations such as: pulp dentin complex protection (lining); auxiliary anodyne curative in the recover of inflamed pulp; temporary filling; Root filling; temporary or permanent cementing agent for inlay, crowns or bridges and Surgical cement.

TECHNICAL INFORMATION:

EUGENOL is obtained from oil of cloves. It's a colorless or pale yellow liquid.

It darkens and thickens on exposure to air. The product is practically insoluble in water. It is soluble with alcohol, chloroform, ether and oils.

It has an anodyne action (calming) on the pulp, so it's well recommend as a temporary curative restorations. It has balsamic, antiseptic, anti-inflammatory and healing characteristics.

USE INSTRUCTIONS:

As Zinc Oxide - **EUGENOL** cement:

Dispense 2 or 3 drops of **EUGENOL** in a flat glass plate and adding Zinc Oxid until a desired consistency is obtained mixing them.

SPECIAL CARE:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

EUGENOL should not be used in patients is know to have sensibility to the product;

EUGENOL must not be used together with composite material, once it interferes in the polymerization; Spice and burning sensation, when in contact with mucous;

EUGENOL can cause skin irritation;

In patients that consider **EUGENOL** taste extremely disagreeable and in cases that surgical cement is used for many weeks, it can results a gastric disturbs;

Highly contra-indicated for pregnants, once it provokes contractions in the uterus.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY