CLARIDEX EVOLUTION 10% / 16% / 22%

Dental Bleaching on Carbamide Peroxide

biodinâmica®

CONTENTS:

Refill: 1x3g syringe **CLARIDEX EVOLUTION 10% or 16% or 22%**; 1 application tip.
Economical kit: 3x3g syringes **CLARIDEX EVOLUTION 10% or 16% or 22%**; 3 application tips.
Kit: 5x3g syringes **CLARIDEX 10% or 16% or 22%**; 5 application tips.
ANVISA nº 10298550056.

INDICATION:

CLARIDEX EVOLUTION is a professionally administered at-home tooth whitening system, indicated for vital teeth, diagnosed, indicated and attended by the dentist.

COMPOSITION:

Carbamide Peroxide 10%, 16% or 22%; Carbopol; Sodium Fluoride; Potassium nitrate; Preservative; Sweetener; Stabilizer; Neutralizing; Humectant and deionized water.

TECHNICAL INFORMATION:

CLARIDEX EVOLUTION is a high viscosity carbamide peroxide gel, neutral pH (6,0 to 7,0) administered at home and attended by a professional. It's uniformly applied in individual trays positioned and fixed in order to keep teeth in contact with the gel. It can be used during the day (when occurs larger formation of saliva) for 2 to 4 hours, or for 8 to 10 hours during the night, depending of the dentist indication. The results of **CLARIDEX EVOLUTION** usually can be noticed after the 3rd or 4th day of use. The treatment takes 10 to 14 days on average although the duration will depend on the teeth natural color, the kind of stain and the severity degree. **CLARIDEX EVOLUTION 10%** is indicated for natural stains deriving from age, tobacco, or in case of the presence of dental sensibility. **CLARIDEX EVOLUTION 16% and 22%** is indicated in older and difficult removal stains or to obtain the results in a short period of time. Staining the more difficult seems to be it's removal. On tetracycline and dentine staining or fluorosis, the oxidation requests more time and the result will depend on the severity degree. Metallic pigmentation stains (amalgam and some endodontic sealer cements) are difficult to be removal. On the necessity of replacement the existing aesthetic restorations, with their original color modified by whitening action, it is recommended to wait 7 to 15 days after bleaching, to occur the final color stabilization and also to eliminate any sign of oxygen derivatives, which can interfere on the polymerization of adhesive systems and composites used on restored treatment. As prevention, no ingestion of alcoholic drinks is recommended and the smoker patients must do abstinence of tobacco derivatives during all the bleaching treatment. If these actions are not possible, select another kind of whitening treatment. Deionized water is the stabilized vehicle for the active ingredient, carbamide peroxide, which acts on the tooth structure, promoting whitening. Sodium Fluoride and Potassium Nitrate act as desensitizing agents. The mint essence gives to the product a pleasant flavor. Carbamide peroxide release its free radical that act as oxidizing agent with the penetration of oxygen inside enamel and dentin structures by permeability. The bleaching agent's action is related with its penetration inside dental structures (enamel and dentin) and with its reaction with pigmented molecules (macromolecules), oxidizing and making them simple, and less pigmented.

SALIVA SOLUBILITY: The product when in contact with saliva is degraded by the salivary peroxidases. The clinical function of salivary enzymes is to raise the conversion of carbamide peroxide into oxygen, improving the effectiveness of bleaching treatment. Peroxide concentration in the first minutes after application is 10% or 16%, decreasing as degradation occurs.

SPECIAL CARES:

BEFORE BLEACHING TREATMENT:

1- Inspect oral environment to ensure appropriate conditions to start treatment like: healthy gingival tissue, bacterial plaque control, caries-free teeth, infiltration in restorations or other conditions that can interfere in bleaching treatment. 2- Treat the dentin hypersensitivity if necessary, during the whitening treatment the exacerbation of the pain is common. 3- Register the teeth color with a shade guide and/or photographs, assess and inform clearly patient about the results can be expect. 4- Mold both arch with alginate, proceed tray fabrication, try the trays and starting with the superior arch. 5- Make adjustments of the trays and demonstrate for the patient the quantity to be used. (ATTENTION: BE SURE INSTRUCT THEM TO NOT EXCESS GEL TO AVOID CONTACT OF PRODUCT WITH GINGIVAL TISSUES). 6- Inform to the patient how the product must be used and give the use instructions to the patient, asking for the complete comprehension of the material before starting the bleaching process. 7- Ask the patient to return every 3 days, mainly at the beginning of the treatment, to evaluate the process. 8- Inform about adverse events: 1. Oral irritation (gingiva, tongue and lips), occurred by the contact of gel with these tissues (explain the importance of adequate quantity to avoid excess out of tray). 2. Cold sensitivity that disappears in 1 to 3 days after the interruption or at the end of the treatment. 3. Uncertain sensitivity, discomfort, impossibility of use or allergic reaction, discontinue treatment and contact your dentist. 4. A darkness in the tray can occur where existing amalgam restoration.

In case of general sensitivity, use always **CLARIDEX EVOLUTION 10%**. For sensibility due to exposition of cervical dentin by gingival retraction, use **CLARIDEX EVOLUTION 10%** and protect the sensitive area, cutting the tray in such a way to prevent the contact of product with the sensitive area. Wait 1 to 2 days and if the sensitivity remains, ask the patient to use the product on alternate days, each 2 or 3 days remaining with the tray only for 1 to 2 hours.

AFTER BLEACHING:

1. Check sensitivity, and if necessary, treats appropriately using indicated desensitizing. 2. Stimulate the patient to maintain the whitening effect obtained by treatment, explaining the importance of dental visits, for evaluations and reinforcement sessions if necessary.

PATIENT'S USE INSTRUCTIONS:

1. Before each application floss and brush your teeth, carefully avoiding hurting the gingiva. Food deposits can interfere on the

result of the whitening treatment.

2. Place a small quantity of **CLARIDEX** in the deeper part of the tray, uniformly.

3. Place the tray with **CLARIDEX** in the mouth pressing slightly to involve the teeth well with the gel and be sure that the tray is completely seated.

4. Remove exceeding gel using gauze or humid paper towel.

5. Use tray according to the regimen of use indicated by your dentist.

6. After used during recommended period of time, remove the tray and brush your teeth. 7. Clean the tray carefully using a dental brush to remove remaining product and keep it in a clean, fresh, airing place.

PATIENT'S COMPLEMENTARY INFORMATION:

There are other types of bleaching treatment, as in office treatment with the use of light activated products. Use **CLARIDEX EVOLUTION** only under the supervision of a dentist following all instructions. Avoid ingestion of food during 40 minutes after using **CLARIDEX EVOLUTION**. Only water is allowed to drink. Coffee, tea, beverage on Cola basis and food that contain coloring (that usually pigment the top of the tongue) must be avoided during the treatment. Do not smoke or drink alcoholic beverages during all treatment. Do not use **CLARIDEX EVOLUTION** on pregnancy or breast-feeding. Store the product in a cool place, protected from direct sunlight and humidity, in its original packaging, at temperatures between 29°C and 28°C. After use keep syringes tightly close to avoid exposure to light.

PRODUCT ADVANTAGES

-Simple application

-The treatment is practical, and does not interrupt the daily routine of the patient

-Allow the application on specific teeth or areas.

PRODUCT DISADVANTAGES

-The incorrect use of the product by the lack of indication and/or professional monitoring and no observation of the use instructions

SPECIAL CARE:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

CLARIDEX EVOLUTION should not be used in the situations below:

- Patients who can't avoid smoking or consuming alcoholic drinks during the treatment. - Darkness tooth caused by pulp injury;

-Cases of teeth darkening caused by pulp injury. - Recent periodontal surgery patients. - Pregnant or breast-feeding women.

- Patient is known to have sensibility to any of the ingredients listed. - Sensitivity teeth. - Exposed root. - Use of medicines for xerostomia. **CLARIDEX** should not be used in patients is known to have sensibility to any of the ingredients listed.

PRECAUTIONS:

-After use, the product packaging must be perfectly sealed to avoid residue on the outside or the material being exposed to light.

-Dispose of the product in accordance with local legislation, ensuring its complete decharacterization in order to prevent its reuse and environmental damages.

WARNINGS:

-Long term use of the product can cause adverse pulp effects.

-It can cause severe irritation and possible damage on gingiva.

-It can cause sensitivity.

-Contact manufacturer if you have any question regarding the product.

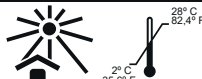
Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

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