EPIC HEAVY FLOW

Resina Composta Nano-Híbrida Fluida

biodinâmica®

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ANVISA No.: 10298559049

COMPOSITION:

Methacrylic monomer matrix; inorganic filler; initiators; stabilizer, preservative and pigments.

INDICATIONS:

Indicated for base, lining, Class I to V restorations, and repair of restorations in deciduous and permanent teeth. It can be used for splinting and in combination with reinforcement fibers for making provisional crowns and bridges.

TECHNICAL INFORMATION:

EPIC HEAVY FLOW is a nano-hybrid, light-curable, low-viscosity composite that allows for filling cavity irregularities and adaptation to preparation walls, even in hard-to-reach areas. Furthermore, it has the ability to absorb the polymerization stresses of the restorative resin. The high filler content in the composition, with organically modified micro and nanoparticles, gives the product greater strength, stability, and shine, with lower polymerization shrinkage than conventional flowable resins. EPIC HEAVY FLOW is a radiopaque restorative material, easy to handle, insoluble in the oral environment, and compatible with all adhesive systems and composite materials. It has lower flowability and higher filler content (between 80% and 84%) which provides mechanical properties similar to regular viscosity composites. The difference between EPIC HEAVY FLOW and conventional flow resins is that EPIC HEAVY FLOW allows for use in various types of procedures and cavities where the professional has the possibility to use a resistant restorative material with precise handling, free of bubbles, and better marginal adaptation.

INSTRUCTIONS FOR USE:

1. Perform prophylaxis with pumice and water; if necessary, mark the occlusal contact points;
2. Select the shade with hydrated teeth, in natural light;
3. Perform the cavity preparation, according to the recommended technique;
4. Perform absolute isolation;
5. Apply the selected adhesive system, according to the selected adhesive technique and the manufacturer's

instructions;

6. Apply in layers of maximum 2 mm for opaque shades and 3 mm for others. For restorations with greater depth, apply and polymerize the material in layers;

7. Light-cure for 40 seconds to ensure complete polymerization of the material;

Note: Position the tip of the light-curing device as close as possible to the material to be polymerized.

8. Perform occlusal adjustment, if necessary, and finishing and polishing.

SPECIAL CARE:

-The dentin-pulp complex must be protected with an appropriate base before acid etching.

-Eugenol-based liners negatively influence the polymerization time.

The use of cationic polywashes, plaque disclosers, and chlorhexidine may interfere with the material's color.

-For satisfactory polymerization, use adequate quality light-curing units, and the distance for polymerization from the restoration surface should be less than 5 mm; strictly observe the exposure time, as well as the material thickness; inadequately polymerized resins can cause irritation to the dentin-pulp complex, as well as reduce mechanical properties and cause color change. During lengthy procedures, deflect the light from the reflector to avoid premature hardening of the material.

-The use of absolute isolation is recommended.

-It is recommended to apply a surface sealer for restorations (e.g., Bioforty) to seal restoration margins and increase the longevity of restorations.

-Dispose of the product in accordance with local legislation, ensuring its complete decharacterization to prevent its reuse and environmental damage.

CONTRADICTIONS:

Contraindicated for persons with reported sensitivity to any of the product components.

WARNINGS:

-Do not use products beyond the indicated expiration date.

-The manufacturer is not responsible for damages caused by incorrect or unintended use, not specified in the instructions for use.

-Keep the product out of the reach of children.

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